


## Quick Start Guide

# Performing a Blood Pressure Check with the SmartMeter Monitor

1

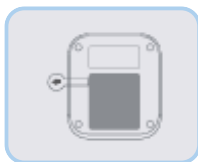

Once batteries are installed, please **pull** the plastic insulating strip before **first** use. Once powered **ON**, the device will automatically establish the initial pairing with the cellular network.

2




While pairing, these symbols will **alternate** on the screen. This process normally takes about **15 seconds**.

3



When pairing is completed successfully, the device will display this symbol and then shut **OFF**. The device is now **ready to use!**

4

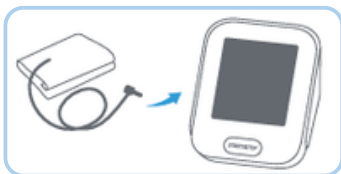

**Plug** the connector on the cuff tube **into** the device. **Be sure to use the port marked 'Cuff Port'** instead of the port for AC adapter (on the opposite side).


5



**NOTE:** Before use, please confirm the cuff fits your arm circumference and is not too loose or tight. Multiple cuff sizes are available - pediatric, standard, XL and XXL. If your cuff does not fit properly, please contact WCR for a replacement.

Roll or push up your sleeve to expose your **upper** arm. Hold your arm with your palm facing **UP** and apply the cuff, align the air tube toward the **center** of your arm.

6

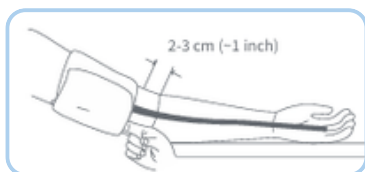

Wrap the end of the cuff over your arm to secure it in place. **Don't make it too tight— allow a finger to fit between the cuff and your arm.**

**Please Note:** This device is intended for use under medical supervision from your physician and results will be monitored by your LVAD care team. Do not make any changes in your treatment plan without consulting your physician for direction. If you are unable to reach your physician or LVAD care team, for any medical emergencies, call 911 or go to the nearest emergency room.

**Disclaimer:** Quick start guide instructions reference the SmartMeter manufacturer Instructions For Use (IFU). A complete instruction Manual is included with each device for full reference and additional information can be found at <https://smartmeterrpm.com/wp-content/uploads/2025/04/iBloodPressure-Classic-Multicuff-User-Manual-ENG-03.10.25.pdf>.


Need Help or Technical Assistance? Call 833.WCR.6364 (833-927-6364)

©Bravida Medical Company 062025  
Intellectual property requires written permission for reproduction or use.

## Quick Start Guide

# Performing a Blood Pressure Check with the SmartMeter Monitor

7

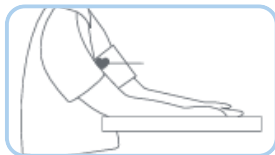

Place your arm resting **comfortably** on a **flat** table. The cuff worn on your arm should be placed at heart level.

8



Press the "**START/STOP**" button to turn **ON** the monitor, the cuff will automatically **inflate**, take a reading and then **deflate**. Remain **still** until the cuff deflates completely.

9



Your results will be displayed and securely stored on the device prior to transmission starting.

10



During data transmission the cloud symbol will **flash**. This process normally takes around **5 to 10** seconds.

11



If **successful**, the LCD will display **OK**. If unsuccessful, the LCD will display "**E5**" or "**E6**", and the measurement will be sent on the **next** successful connection.

**Please Note:** This device is intended for use under medical supervision from your physician and results will be monitored by your LVAD care team. Do not make any changes in your treatment plan without consulting your physician for direction. If you are unable to reach your physician or LVAD care team, for any medical emergencies, call 911 or go to the nearest emergency room.

**Disclaimer:** Quick start guide instructions reference the SmartMeter manufacturer Instructions For Use (IFU). A complete instruction Manual is included with each device for full reference and additional information can be found at <https://smartmeterrpm.com/wp-content/uploads/2025/04/iBloodPressure-Classic-Multicuff-User-Manual-ENG-03.10.25.pdf>.

Need Help or Technical Assistance? Call 833.WCR.6364 (833-927-6364)

©Bravida Medical Company 062025 Intellectual property requires written permission for reproduction or use.