

Quick Start Guide

iDigiTemp SmartMeter Thermometer Quick Start Guide

Ear Mode

1



Clean Probe

2



Pull Ear
Back & up

3



Insert Probe
Insert the probe gently
to take measurement

Guidance



Keep Steady
Keep still during your
measurement



Repeated Use
Wait 10 seconds between
each use.



Operating Temperature
10-40 °C
50-104 °F



Keep Probe Clean
After continuous use,
make sure to check and
clean the lens.



Operating Environment
Make sure the user and
thermometer are in the same area
for at least 30 minutes prior to use.



Cleaning
After cleaning the
thermometer, wait 15
minutes before using.

Please Note: This device is intended for use under medical supervision from your physician and results will be monitored by your LVAD care team. Do not make any changes in your treatment plan without consulting your physician for direction. If you are unable to reach your physician or LVAD care team, for any medical emergencies, call 911 or go to the nearest emergency room.

Disclaimer: Quick start guide instructions reference the SmartMeter manufacturer Instructions for Use (IFU). A complete instruction Manual is included with each device for full reference and additional information can be found at <https://smartmeterrpm.com/wp-content/uploads/2026/01/iDigiTemp-User-Manual-10.28.25.pdf>.

Need Help or Technical Assistance? Call 833.WCR.6364 (833-927-6364)

©Bravida Medical Company 062025
Intellectual property requires written permission for reproduction or use.

Quick Start Guide

iDigiTemp SmartMeter Thermometer Quick Start Guide

Connecting to the Cellular Network




When the device is turned on for the first time after battery **installation** or battery **replacement**.



The device will automatically scan.



And connect to the cellular network.

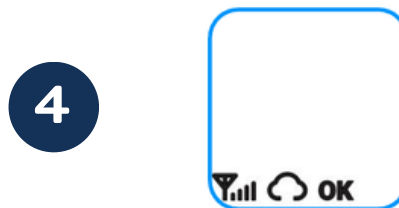


The symbol **OK** will appear on the screen, confirming the connection was **successful**.

Measurement Data Upload



After each measurement, the measurement data will be automatically transmitted.

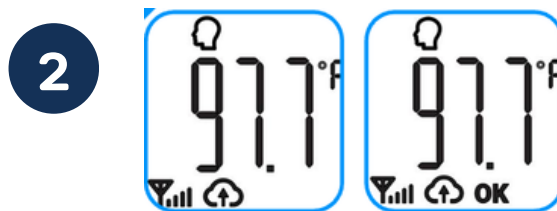

Wait for the **OK** to appear before putting the cap back on for the measurement to transmit via cellular network.

Please Note: This device is intended for use under medical supervision from your physician and results will be monitored by your LVAD care team. Do not make any changes in your treatment plan without consulting your physician for direction. If you are unable to reach your physician or LVAD care team, for any medical emergencies, call 911 or go to the nearest emergency room.

Disclaimer: Quick start guide instructions reference the SmartMeter manufacturer Instructions for Use (IFU). A complete instruction Manual is included with each device for full reference and additional information can be found at <https://smartmeterrpm.com/wp-content/uploads/2026/01/iDigiTemp-User-Manual-10.28.25.pdf>.

Need Help or Technical Assistance? Call 833.WCR.6364 (833-927-6364)

©Bravida Medical Company 062025 Intellectual property requires written permission for reproduction or use.